lionelo
care

User manual
Instrukcja obsługi
Bedienungsanleitung
Инструкция по эксплуатации
Manuale d'uso
Manuel de l'Utilisateur
Manual de usuario
Handleiding

Naudojimo instrukcija
Návod k obsluze
Használati utasítás
Manual utilizare
Bruksanvisning
Bruksanvisning
Brugsvejledning
Käyttöopas

Twee Free

Breast pump

Laktator

Muttermilchpumpe

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



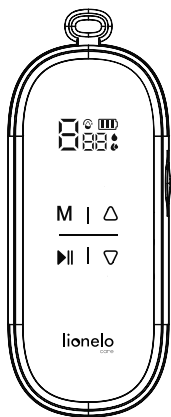
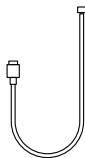
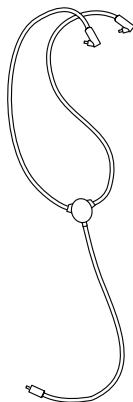
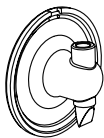
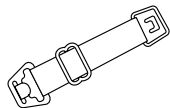
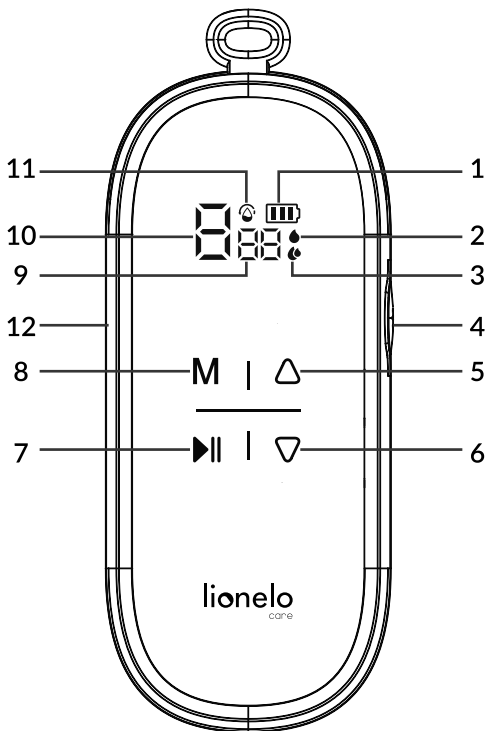
B

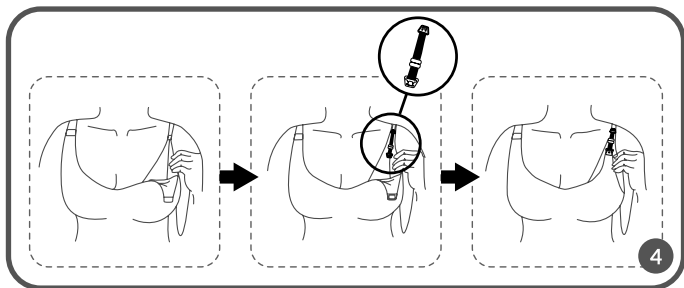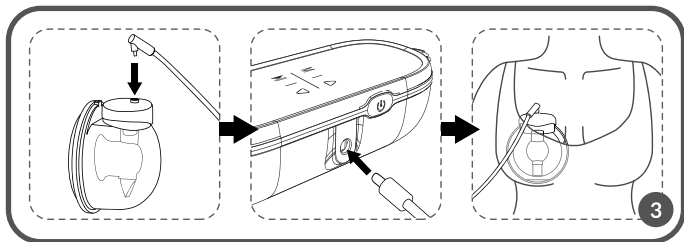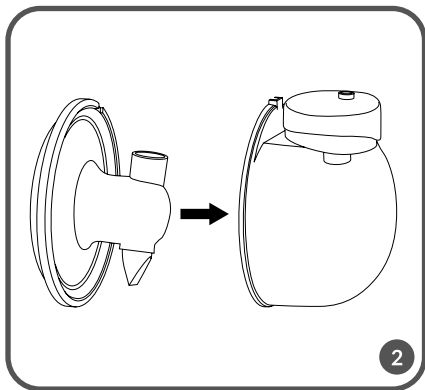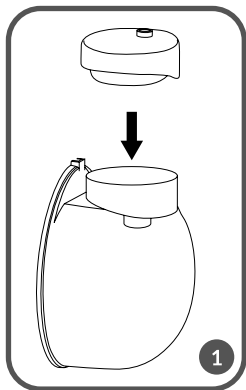Dear Customer!

In case of any questions or comments on the purchased product, please contact us:

help@lionelo.com

Producer:

BrandLine Group Sp. z o. o.

A. Kręglewskiego 1, 61-248 Poznań, Poland

Permissible operating temperature of the device: 5 - 45°C.

IMPORTANT INFORMATION

Please read this instruction manual before operating the product to familiarise yourself with its functions and use it as intended.

Failure to follow the safety instructions and use not in accordance with the instruction manual may result in damage to your health. Keep this instruction manual for future reference.

- 1.** This device may be used by children and persons with reduced physical, sensory or mental abilities, as well as by persons without relevant knowledge and experience in the use of the device, if they are supervised or have been previously instructed in the safe use of the device and the risks involved.
- 2.** This device is not a toy. Do not allow children to play with it. Keep the device out of the reach of

children. Cleaning and maintenance must not be performed by children without adult supervision.

3. Do not repair or modify the device. This may only be undertaken by an authorised service centre. Use only original spare parts.
4. Do not operate the device if you have noticed any damage. This applies to both the device itself and its accessories, e.g. the USB cable.
5. Do not use the device if it begins to malfunction, has been dropped or submerged in water, becomes excessively hot, discolouration, bulging or other abnormal phenomena occur. Contact the manufacturer's authorised service centre.
6. Never leave the device connected to a power socket unattended.
7. The device is not resistant to high temperatures. Use the device away from heat sources, high temperatures, hot surfaces, direct sunlight, sources of spark, open flames, oils and sharp edges.
8. Do not use chemicals to clean the device. Do not wash the main device with water.
9. Switch off the device after each use.
10. Do not immerse the entire device in water or other liquids. Do not use near water.
11. Do not misuse the device.
12. Do not use the device or touch the power source with wet or damp hands. Make sure all breast pump parts are dry before assembling the device.

- 13.** Before cleaning or moving the device, switch it off and disconnect it from the power source.
- 14.** Before and after each use, wash and disinfect all parts that will adhere to one's body and come into direct contact with milk.
- 15.** Thoroughly dry each component before use and storage.
- 16.** For hygienic reasons, the device should be used by one person.
- 17.** If using the breast pump causes discomfort or pain, stop using the device immediately.
- 18.** Do not use the device throughout your pregnancy.
- 19.** Do not use the device when tired or while sleeping.
- 20.** Special care is required when using the device near children.
- 21.** Do not use the device while lying down or during strenuous exercise or activities that may cause injury.
- 22.** If breast milk does not appear for more than 5 minutes while using the device, stop using the breast pump and consult your doctor.
- 23.** If you have an implanted pacemaker or other electrical device, it is recommended that you consult your doctor before using the device.
- 24.** If you use the device at the highest permissible ambient temperature for 30 minutes, the silicone insert may reach a high temperature.
- 25.** It is recommended to replace the components in

contact with the milk every approx. 3-6 months, depending on the frequency of use, and when the device has not been used for a long time.

26. Do not expose the device and its batteries to extreme temperatures. The permissible temperature range in which the battery can be charged is 5°C - 35°C.
27. When the battery is fully charged, immediately unplug the device from the power socket.
28. Do not use milk from bottles or containers that have signs of damage on them.
29. The batteries in this device are not replaceable.
30. The use of a power supply with a maximum rating of 5.0V DC 1.0A is recommended.

SET (FIG. A)

- | | |
|-----------------------|-------------------------------|
| 1. Main unit x1 | 5. Food container (150 ml) x2 |
| 2. USB cable x1 | 6. Silicone membrane x2 |
| 3. Connecting tube x2 | 7. Membrane cover x2 |
| 4. Silicone funnel x4 | 8. Bra clasp x2 |

DIAGRAM OF THE DEVICE (FIG. B)

- | | |
|--------------------------|-------------------------------|
| 1. Battery level | 7. Pause button |
| 2. Massage mode | 8. Mode change button |
| 3. Breast pumping mode | 9. Operating time indicator |
| 4. On/off switch | 10. Intensity level indicator |
| 5. Power increase button | 11. Mixed mode |
| 6. Power decrease button | 12. USB port |

BREAST PUMP MODES

- Massage mode – 9 levels. Massage simulates the initial sucking phase of the baby during breastfeeding.

- Breast pumping mode – 9 levels. This mode simulates the deeper sucking phase of the baby during breastfeeding.
- Mixed mode – 9 levels. Combines massage and breast pumping modes to simulate the baby's final sucking phase. Used for pumping leftover milk.

To change mode, press the mode change button on the display (8, fig. B).

BREAST PUMP INSTALLATION

Make sure all components are dry, clean and disinfected before installing the device.

1. Place the silicone membrane (6, fig. A) inside the cover (7, fig. A). Then place the whole inside the food container (fig. 1).
2. Place the silicone funnel (4, fig. A) on the food container (fig. 2). Make sure it is fitted correctly and fits tightly to the container without standing out at any point.
3. Sit up straight, lean forward slightly and make sure the device fits snugly on the breast and there is no space between the breast and the funnel.
4. Connect the food container to the main device using the connecting tube (fig. 3).
Note! If you have a stiff and inflexible bra, use the bra clasp provided (8, fig. A). To do this, unhook the bra strap, then attach the clasp and adjust the length according to your preference (fig. 4).

USE

Note! Before first use, charge the device (see: Charging).

To switch the device on, press and hold the switch (4, fig. B). To switch the device off, press the switch (4, fig. B). The device will automatically switch off after approx. 30 minutes of operation.

INTENSITY SETTINGS

The device has 9 intensity levels, which can be adjusted using the intensity decrease and increase buttons (5,6, fig. B).

The intensity adjustment is available in each of the device's operating modes. The selected intensity level (from 1 to 9) will appear on the display (10, fig. B).

PAUSE FUNCTION

Press the pause button (7, fig. B) to interrupt the operation of the device. Press again to resume operation.

The pause lasts for a maximum of 5 minutes. After this time, the device will switch off automatically.

CHARGING

Note! The device has a built-in lithium-ion battery. The use of a power supply with a maximum rating of 5.0V DC 1.0A is recommended. It is recommended to use the USB cable included in the set.

The battery level is indicated by the icon (1, fig. B) on the display. A low battery level will be indicated by a flashing icon.

The device can be used while charging. Special care is recommended – frequent use of the device while charging may result in reduced battery life.

STORING AND HEATING MILK

Storage

- At room temperature (16-25°C), milk can be stored for up to 4 hours (up to 6 hours in the event of very clean collection conditions).
- In a refrigerator (4°C), milk can be stored for up to 3 days (up to 5 days in the event of very clean collection conditions).
- In the freezer (-18°C), milk can be stored for up to 6 months (up to 9 months in the event of very clean collection conditions).
- Do not store milk on the refrigerator door.

Defrosting and reheating

- Freeze milk in food-grade plastic bottles or milk storage bags.
- To freeze milk, fill the container $\frac{3}{4}$ full, as milk expands in volume when frozen.
- To preserve the nutritional value of the milk, defrost it naturally in the fridge (up to 24 hours) or at room temperature (up to 2 hours). You can also defrost milk under warm running water at a maximum temperature of 37°C. The milk should then be used immediately after defrosting.
- Before feeding, shake the container with the milk gently to mix it evenly.

NOTE

- Always check the temperature of the milk before feeding.
- Do not defrost frozen milk in the microwave or boiling water to avoid nutritional loss and scalding your baby.

- Never re-freeze milk that has already been defrosted and store it either at room temperature or in the refrigerator.

CLEANING AND DISINFECTION OF THE DEVICE

NOTE

- Do not clean or disinfect the main body or the power cord of the device. These parts do not come into contact with food.
- Wash all parts in contact with body and milk separately immediately after use to avoid drying out food residues and to prevent the growth of bacteria. Do not wash any parts of the device in the dishwasher.

Before first use

Before using the device for the first time, wash with detergent and then sterilise with boiling water all its parts that will come into contact with milk (milk container, tee, valve, diaphragm, and funnels), except the main body. Follow the cleaning and disinfection instructions below.

After each use

- After use, disassemble the main body and clean it with a dry and soft cloth.
- After each use, disassemble the device into its individual parts and rinse with cool water all separated parts in contact with the body and with the milk to remove food residues.
- Wash all separated parts in warm water with a mild detergent and then rinse thoroughly under clean running water.
- To disinfect, place all parts in boiling water for at least 10 minutes. Make sure that all parts are immersed in the water and none is directly touching the bottom of the pot.
- Leave all device parts to dry on a clean towel or paper towel.
- Store the device parts in a clean bag, container or other clean place until the next use. If the parts are still wet, do not store them in a sealed container.

NOTE

- Clean parts that have been in contact with your body once a day and after each use. Be careful not to damage the device when cleaning it.
- Do not use sharp tools to clean the silicone parts, as this may result in damage.

- Take care not to damage the suction valve and the silicone diaphragm when cleaning. If these are damaged, the device will not function properly.
- Remove the suction valve from the device carefully. When cleaning it, rinse it in clean water without inserting your finger or any objects into it, as this may result in damage.
- Do not use disinfectants to clean the device.
- If, after sterilisation in water, you notice a white deposit on parts of the device, this may indicate that the water used is hard. Wipe off the residue using a clean towel and leave the parts to dry on their own.

The photos are for reference only, the actual appearance of the products may differ from the photos.

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EN: The product complies with the requirements of European Union directives. In accordance with Directive 2012/19/EU, this product is subject to separate collection. The product should not be disposed of with municipal waste as it may pose a risk to the environment and human health. Return the used product to the recycling centre for electrical and electronic equipment.

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